

# K70006 KEY-COAG

## INTENDED USE/ SUMMARY:

Lyophilized rabbit plasma is intended for the tube coagulase test to assist in the identification of *Staphylococcus aureus*. Coagulase from *Staphylococcus aureus* binds plasma fibrinogen, causing the organisms to clot plasma. Two different forms of coagulase, free and bound, can be produced by *Staphylococcus aureus*. The tube test can detect the presence of both bound and free coagulase, therefore the isolates that do not produce the bound coagulase (clumping factor) must be tested for the ability to produce extracellular coagulase (free coagulase).

## FORMULA:

KEY-COAG contains lyophilized rabbit plasma in 0.5 ml single use aliquots.

## STORAGE AND SHELF LIFE:

Storage: Store vials at 2-8 degrees C. The product may be shipped without refrigeration if transit time is less than one week.. Product should not be used if there are any signs of deterioration, contamination, or if the expiration date has passed. The expiration date applies to the product in its intact packaging when stored as directed.

## PRECAUTIONS:

For *invitro* diagnostic use by adequately trained and qualified laboratory personnel only. Observe established procedures against microbiological contamination / hazards when performing the test. Discard in an appropriate manner for biohazard material in accordance with institutional guidelines and good laboratory practices.

## PROCEDURE:

Re-hydrate the KEY-COAG tube with 0.5 ml or 10 drops of sterile distilled water. Shake gently for 30 seconds to dissolve the plasma. Inoculate with a loop full of the test organism and incubate the tube at 35° C for four hours without shaking. Observe hourly during incubation since fibrinolysin produced by some staphylococci could lyse the clot, thus giving a false negative result.

## INTERPRETATION OF RESULTS:

Clot formation is considered a positive coagulase test.

## LIMITATIONS:

Coagulase positive organisms should be examined by gram stain to determine morphology and gram reaction, and tested for catalase activity. Additional tests are recommended for complete identification.

## MATERIALS REQUIRED BUT NOT PROVIDED:

Standard microbiological supplies and equipment such as sterile water, loops, other culture media, sticks, incinerators, and incubators, etc., as well as serological and biochemical reagents, are not provided.

## QUALITY CONTROL:

Minimal quality control requires testing with any known coagulase negative and coagulase positive organisms at the time test is performed. Key regularly uses:

- *Staphylococcus aureus* ATCC 25923 Results: Clot formation within 4 hours
- *Staphylococcus epidermidis* ATCC 12228 Results: No clot formation

## REFERENCES:

1. Baron, E.J., L.R. Peterson and S.M. Finegold, 1994, Bailey & Scott's diagnostic microbiology, 9th ed. Mosby-Year Book, Inc., St. Louis, MO.
  2. Murray, P.R., E.J. Baron, M.A. Pfaller, F.C. Tenover and R.H. Tenover, 2007. Manual of clinical microbiology, 9th ed. American Society for Microbiology, Washington, D.C.
- ATCC is a registered trademark of the American Type Culture Collection.



KEY SCIENTIFIC PRODUCTS, INC.  
1113 EAST REYNOLDS STREET  
STAMFORD, TEXAS 79553  
VOICE 800-843-1539  
FAX 888-440-4208  
WWW.KEYSCIENTIFIC.COM

K70006-0613