

## Safe-Mount K70010

### Intended Use:

Safe-Mount is an alcohol-based mounting medium intended for creating permanent records of your Gram stain from clinical specimens or cultures, hematology smears and cytology slides. This product does not contain toxic compounds such as toluene, xylene or dibutylphthalate (DBT). Slides examined under oil immersion may be adequately mounted. In addition, mounted cover slips may be simply removed by soaking the slide in distilled water for an hour. Safe-Mount has a low refractive index and does not interfere with subsequent microscopy. Safe-Mount does not cause leaching of stain during storage.

### Procedure:

For slides with immersion oil, apply 100 % alcohol to a lens/tissue paper and gently wipe off the oil thoroughly without affecting the smear. Apply 1-3 drops, depending on the size of coverslip, and let it dry for 15 minutes.

For slides without immersion oil, place 1-3 drops of Safe-Mount onto the stained slide and apply the coverslip. Dry the mounted slide at room temperature for 15 minutes before storage.

**Storage:** Store between 15-25°C; do not refrigerate.

**Precautions:** This product contains flammable solvent; do not use near a flame. If spilled, simply wash off with water.

Do not wet the rim during use, and after each use secure the dropper-cap tightly to prevent solvent evaporation.

Contents are non-toxic. The product emits a slight alcohol odor.



**KEY SCIENTIFIC PRODUCTS**  
1113 EAST REYNOLDS ST.  
STAMFORD, TEXAS 79553  
[WWW.KEYSCIENTIFIC.COM](http://WWW.KEYSCIENTIFIC.COM)  
Voice 800-843-1539  
Fax 888-440-4208

K70010-1109