

Insert for KPC/Metallo-B-Lactamase Confirm Kit (98006)

Insert for KPC/MBL and OXA-48 Confirm Kit (98015)

REVISION:

DBV0034H

DATE OF ISSUE:

02-04-2013

LANGUAGE:

English

KPC/Metallo- $\beta$ -Lactamase Confirm Kit

KPC/Metallo-beta-lactamase and OXA-48 Confirm Kit

FOR IN VITRO DIAGNOSTIC USE ONLY

**PRODUCT GROUP:** Kits for beta-lactamase identification

**MANUFACTURER:** ROSCO Diagnostica A/S, Taastrupgaardsvej 30, DK-2630 Taastrup, Denmark.

**INTENDED USE:** Tablets are used for qualitative *in vitro* identification of microbial resistance mechanisms by the agar tablet/disc diffusion method, in order to confirm the mechanism by which the organism has gained resistance to specific antimicrobial agents. The kit is intended for detection of the carbapenemases; KPC, MBL and OXA-48 in **Enterobacteriaceae**.

**INTENDED USERS:** Only to be used by professionals and people trained to work with microbes and disc diffusion testing.

**PRINCIPLE OF THE TEST:**

Four cartridges of tablets containing 10  $\mu$ g Meropenem (diffusible amount) alone and in combination with inhibitors of different  $\beta$ -lactamases.

Inhibitors are added to differentiate between isolates with and without resistance mechanisms (see explanation below). In addition the KPC/MBL and OXA-48 Confirm Kit (98015) also contains one cartridge of 30  $\mu$ g Temocillin tablets to detect OXA-48 or similar producing isolates.

If an organism shows reduced susceptibility to Carbapenems there can be four likely reasons:

1. The organism hyper-produces AmpC. Because of the slow hydrolysis of carbapenems by the AmpC enzyme, the AmpC is probably coupled to other resistance mechanism like efflux pumps, porin loss or other  $\beta$ -lactamases. The AmpC enzyme is inhibited by Cloxacillin. The Cloxacillin is used to distinguish between AmpC and KPC since both are inhibited by Phenylboronic Acid. Thus a difference ( $\geq 5$ mm) in zones between Meropenem and Meropenem + Cloxacillin indicates AmpC activity.
2. The organism produces a Metallo  $\beta$ -lactamase that hydrolyses carbapenems efficiently. MBLs are inhibited by Dipicolinic Acid and a difference in zone size ( $\geq 5$ mm) between Meropenem and Meropenem + DPA indicates the presence of a MBL. DPA has no (as opposed to EDTA) intrinsic antimicrobial activity and thus the results with this compound are more easily interpret.
3. The organism produces a KPC enzyme. KPC enzymes are inhibited by Phenylboronic Acid. However, Phenylboronic Acid also inhibits the AmpC and in order to raise the specificity of the Kit, the Cloxacillin combination is included to distinguish between the two. So a zone difference ( $\geq 4$ mm) with Meropenem + Phenylboronic Acid but no difference ( $<4$ mm) with the Meropenem + Cloxacillin indicates the presence of a KPC enzyme.
4. The Enterobacteriaceae produce an oxacillinase (OXA-48 or similar). Negative results of all synergy tests, and no zone of inhibition with Temocillin 30  $\mu$ g is presumptive of an OXA-48 or similar. Besides, these isolates are highly resistant to Piperacillin + tazobactam. Please Notice: if both Meropenem and all combinations show no zone of inhibition, the Temocillin test is invalid, and the result inconclusive. The Temocillin test is only valid for Enterobacteriaceae.

DETAILED INSTRUCTIONS:

ROSCO's detailed Instruction for Use for *Detection of resistance mechanisms* should be available in laboratories working with ROSCO's *Diagnostic products*.

Latest edition of Instruction for Use can be seen in and/or printed out from ROSCO's website [www.rosco.dk](http://www.rosco.dk).

Instructions for Use and User's Guide can be obtained free of charge from your local distributor on request, or from ROSCO Diagnostica A/S:

E-mail: [info@rosco.dk](mailto:info@rosco.dk) or

Fax: +45 43 52 73 74

CONTENT AND FORMULATION:

4 or 5 cartridges of tablets, formulated for maximum stability, each containing approximately 50 tablets:

1. Meropenem 10 µg, coded MRP10
2. Meropenem 10 µg + Phenylboronic Acid (KPC and AmpC inhibitor), coded MRPBO
3. Meropenem 10 µg + Cloxacillin (AmpC inhibitor), coded MRPCX
4. Meropenem 10 µg + Dipicolinic acid (Metallo-β-Lactamase inhibitor), coded MRDPDP.
5. Temocillin 30 µg (only in the OXA-48 Confirm kit 98015)

STORAGE/HANDLING:

Store at 2-8 °C in the box provided or unopened cartridges until the expiry date shown on the product label. Allow the cartridges to acclimatize to room temperature for 30-60 minutes before the lid is removed from the cartridge. Once a cartridge has been opened and in particular when placed in a dispenser, it should be kept at room temperature for up to 2 months. If necessary, when in use for a longer period of time than 2 months, the cartridges can be stored at 2-8 °C. Always seal the cartridges with the original green lid, and never place the dispenser in the refrigerator. When stored at 2-8 °C the cartridges should be allowed to acclimatize, as described above, before use.

PRECAUTIONS:

For *in vitro* diagnostic use only. Safety precautions should be taken and aseptic techniques used when working with potential biohazards. To be used only by adequately trained and qualified laboratory personnel. Sterilize all biohazard waste before disposal. Refer to Product Safety Data Sheet.

MATERIALS REQUIRED BUT NOT PROVIDED:

Standard microbial equipment such as loops, culture media, incubator etc. and biochemical reagents.

PROCEDURE:

1. Using a fresh, pure culture prepare a suspension of the organism to be tested equivalent to McFarland 0.5
2. Using a sterile swap or Drigalski spatula spread the suspension uniformly over the entire area of a Mueller Hinton susceptibility agar plate.
3. Using a single tablet dispenser, place one of each tablet on the inoculated agar plate, ensuring sufficient space between individual tablets to allow for proper measurement of inhibition zones. Notice that more than one Confirm Kit can be tested on the same plate.
4. Incubate at 35±1°C for 18±2 hours (overnight).
5. Measure and record the diameter of the inhibition zones. No zone around a tablet corresponds to a 9 mm inhibition zone.

## INTERPRETATION OF RESULTS:

The results are interpreted by comparing the inhibition zones of the different tablets

1. Compare the zone of inhibition of the Meropenem 10 µg tablet to the zones of inhibition of each of the Meropenem + inhibitor tablets. If all zones are within 3mm of each other, record the organism as neither expressing KPC nor MBL activity.
2. Measure the inhibition zone around Meropenem 10 µg (MRP10) and compare to the zones around the two combination tablets; Meropenem + Cloxacillin (MRPCX) and Meropenem + Phenylboronic (MRPBO):  
If the zone around MRPCX is  $\geq 5$ mm, and the zone around MRPBO is  $\geq 4$ mm in comparison to the single disc respectively, the organism is demonstrating AmpC activity alone. The AmpC is probably hyper-produced and/or coupled with porin loss and/or efflux pumps.
3. Measure the inhibition zone around Meropenem 10 µg (MRP10) and compare with the zones around Meropenem + Phenylboronic Acid (MRPBO) and Meropenem + Cloxacillin (MRPCX):  
If the zone around MRPBO is  $\geq 4$ mm and the zone around MRPCX is  $\leq 3$ mm in comparison to the single disc, the organism demonstrates KPC activity.  
*And/or*  
Compare the zones around the two combination discs MRPBO and MRPCX:  
If the zone around MRPBO disc is  $\geq 4$  mm than the zone around the MRPCX disc then the isolate is KPC positive.
4. Measure the inhibition zone around Meropenem 10 µg (MRP10) and Meropenem + DPA (MRPDP):  
If the zone around MRPDP is  $\geq 5$ mm in comparison to the single disc, the organism is positive for Metallo- $\beta$ -Lactamase activity.  
Test only Ceftazidime resistant isolates. False MBL positive may be obtained with Ceftazidime sensitive isolates.
5. Look at the zone around Temocillin 30 µg NeoSensitabs. If there is no zone of inhibition, the strain is presumptively OXA-48 (or similar) positive. In 80% of the cases OXA-48 is accompanied by a CTX-M, ESBL. It can be detected using CAZ/Clavulanate or Meropenem + Tazobactam.
6. It is possible for an organism to be positive for more than one resistance mechanism. So if, for example, if the value in step 4) is  $\geq 5$ mm and the value in step 3) is  $\geq 4$  mm, the organism is both positive for MBL and KPC activity, although in several cases the MBL may mask the KPC making it difficult to detect in the presence of an MBL. No combination of resistance mechanisms is impossible and more than one Confirm kit can be tested on the same plate.
7. Use table 1 to assist in the interpretation  
Isolates processing both KPC and MBL in the same isolate have been described in Greece and Germany. Rosco Diagnostica has developed a triple combination tablet: Meropenem + Phenylboronic + Dipicolinic that permits the identification of both enzymes (inhibition zones compared with Meropenem + DPA and Meropenem + Phenylboronic, respectively)  
The triple disk (68912) is available for detection of KPC+MBL in the same isolate (table 2).

## QUALITY CONTROL:

Although ROSCO Diagnostica A/S produces, by far, the most stable diffusion discs (tablets) it is necessary to perform regular quality control. This should be done with at least one organism to demonstrate a positive reaction and at least one organism to demonstrate a negative reaction. Zones of inhibition obtained using the combination tablets plus the Carbapenem tablet alone against the negative control (i.e. *E. coli* ATCC 25922), should be within 3 mm.

*As positive Q. C. strains the following may be used:*

*Klebs. pneumoniae NCTC 13438, KPC positive*  
*Klebs. pneumoniae NCTC 13439, MBL positive*  
*Klebs. pneumoniae ATCC BAA-1705, KPC positive*  
*Klebs. pneumoniae ATCC BAA-2146, MBL positive*

*As negative Q.C strain the following may be used:*

*Klebs. Pneumonia ATCC 700603*

Table 1:

		Meropenem + Phenylboronic MRPBO	Meropenem + DPA MRPDP	Meropenem + Cloxacillin MRPCX	Temocilin 30 µg
AmpC + porin loss	Meropenem 10 µg MRP10	≥ 4mm and	≤ 3 mm	≥ 5mm	≥ 12mm
ESBL + porin loss (a)	Meropenem 10 µg MRP10	≤ 3 mm	≤ 3 mm	≤ 3 mm	≥ 12mm
KPC	Meropenem 10 µg MRP10	≥ 4mm	≤ 3 mm	≤ 3 mm	Variable
	Meropenem + Cloxacillin (MRPCX)	≥ 4mm	-	-	-
MβL	Meropenem 10 µg MRP10	< 4mm	≥ 5mm	≤ 3 mm	Variable
OXA-48 and similars	Meropenem 10 ug MRP10	≤ 3 mm	≤ 3 mm	≤ 3 mm	No zone of inhibition
OXA-48 + ESBL (a)	Meropenem 10 ug MRP10	≤ 3 mm	≤ 3 mm	≤ 3 mm	No zone of inhibition

(a) : synergism CAZ / Clavulanate.

Neither AmpC, KPC nor MβL: All zones within 3 mm of each other.

OXA-48 show negative results with KPC+MBL Confirm kit, but it is Temocillin resistant (no zone around Temocillin 30 ug Neo-Sensitabs).

Tabel 2		Triple Disk MER + DPA + BO double inhibitor
KPC	MRPDP	≥ 4 mm
MBL	MRPBO	≥ 4 mm
KPC + MBL	MRPDP MRPBO	≥ 4 mm <u>and</u> ≥ 4 mm

REFERENCES: [www.rosco.dk](http://www.rosco.dk)