

PREVOTELLA SPP.

SENSITIVITY DISC PATTERN: Kanamycin and Vancomycin R: Colistin V. Most fluoresce red and are brown to black colonies on blood containing medium. P. Intermedia is the only indole positive strain. Lipase positive strains (P. loescheii and P. melaninogenica) may be separated from each other by **ARG**.

K1464 NAG- N-acetyl-b-d-glucoaminidase
MBGL--4mu B glucosidase
PRO-Proline naphthlyamide

K1707 AGAL -Alpha galactosidase:
ARG Arginine naphthylamide
K2375 IND spot indole

PIGMENT

