

VOGES-PROSKAUER TESTS

PRINCIPLE /DISCUSSION:

Voges-Proskauer tests demonstrate an organism's ability to convert pyruvate to acetoin. This property is a valuable tool in distinguishing *Klebsiella*, *Enterobacter*, *Ewingella* and *Serratia* from other members of Enterobacteraceae, as well as distinguishing between *Proteus vulgaris* (V-P negative) and *P. mirabilis* (V-P positive).

Key Scientific has found that modifications in the original procedure can produce more clear cut reactions as well as shorten the incubation period. Key's rapid V-P test can be used in place of any standard V-P test. Since the pyruvate to acetoin conversion occurs more efficiently at a slightly acid pH, the tablets contain dextrose to insure adequate acid formation.

ACTIVE INGREDIENTS:

Voges-Proskauer Tablets contain 10 mg. of sodium pyruvate per tablet with dextrose and other inert ingredients. WEE-TABS contain 1 mg. sodium pyruvate.

Voges-Proskauer Tablets do not contain any material known at this time to be hazardous.

MATERIALS REQUIRED:

Voges-Proskauer tests require preliminary growth on appropriate media. Consult a manual such as the Manual for Clinical Microbiology for this information. Each item is sold separately.

K660 Voges Proskauer Base -50/bottle

K670 Voges Proskauer Buffer -50/bottle

K1660 WEE-TAB-28 ready-to-use tubes.

K980670 40% KOH (V-P "B" reagent) -3 mls.

K980660 alpha-naphthol (V-P "A" reagent) -3 mls

The following items are also required but not provided:

Test tubes for large tablets

Loop for colony transfer

Distilled water

PROCEDURE:

(1) Place 1 of each V-P Tablet and 1 ml. of distilled water into a test tube.

(2) Inoculate heavily with either 2-3 drops of a heavy suspension of the organism to be tested or with a heavy loopful of organism directly from the plate. Best results require a final suspension of at least a #3 McFarland.

(3) Incubate at 35-37C for 6-8 hours, but no more than 8 hours. Inoculating the V-P tube and refrigerating overnight for incubation the next day will not adversely affect the test.

(4) After incubation, 2 drops of alpha-naphthol reagent and mix by gently shaking.

(5) Next add add 3 drops of 40% KOH. The reagent will form a thin layer on the surface of the liquid. Allow the tube to stand, up to 30 minutes, observing periodically for the appearance of a cherry red color.

INTERPRETATION:

In a positive V-P test, the acetoin will react with alpha-naphthol in the alkaline environment provided by the KOH to produce a cherry red color. The color develops first at the surface and then spreads gradually into the lower part of the tube. In a strong reaction the color appears almost immediately but weaker ones may take 20-30 minutes. No color change, or the appearance of a copper color is a negative result.

QUALITY CONTROL:

KEY Voges-Proskauer Tablets should be tested prior to use with organisms which produce known reactions. Key Scientific suggests *Enterobacter aerogenes* ATCC 13048 for positives and *E. coli* ATCC 25922 for negative. Finished tests should be discarded in a manner appropriate for biohazardous materials

STORAGE:

Store unused tablets, with desiccant, tightly sealed at room temperature.

USUAL REACTIONS:

<i>Serratia marcescens</i>	+
<i>Serratia liquifaciens</i>	+
<i>Klebsiella pneumoniae</i>	+
<i>Klebsiella oxytoca</i>	+
<i>Enterobacter aerogenes</i>	+
<i>Enterobacter cloacae</i>	+
<i>Proteus mirabilis</i>	+
<i>Proteus vulgaris</i>	-
<i>E. coli</i>	-
<i>Salmonella</i>	-
<i>Shigella</i>	-
<i>Citrobacter freundii</i>	-
<i>Morganella morganii</i>	-
<i>Yersinia enterocolitica</i>	-

REFERENCES:

1) Manual of Clinical Microbiology, Fifth Edition, Chapter 120, "Quality Control" and Chapter 122, "Reagents and Stains"

2) Bailey and Scott's Diagnostic Microbiology, Seventh Edition, Chapter 27, "Enterobacteraceae"



KEY SCIENTIFIC PRODUCTS
1113 EAST REYNOLDS ST.
STAMFORD, TEXAS 79553
WWW.KEYSCIENTIFIC.COM
Voice 800-843-1539
Fax 888-440-4208